



Resolve® Drainage Catheters


LOCKING INSTRUCTIONS

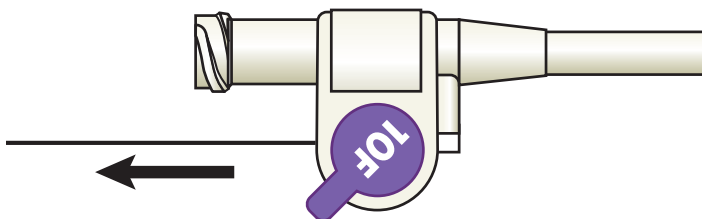

STEP 1: To engage the suture locking mechanism: Pull the suture until desired pigtail is formed.

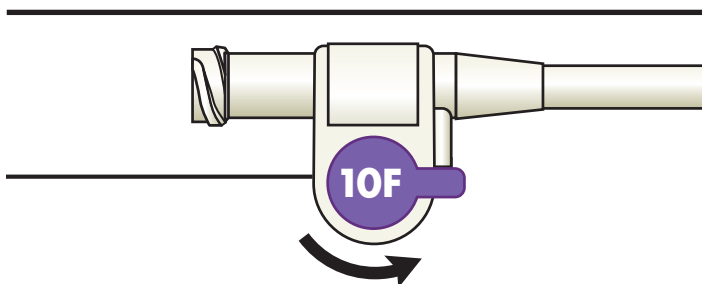

STEP 2: Rotate the suture locking mechanism distally to hold the suture in place.

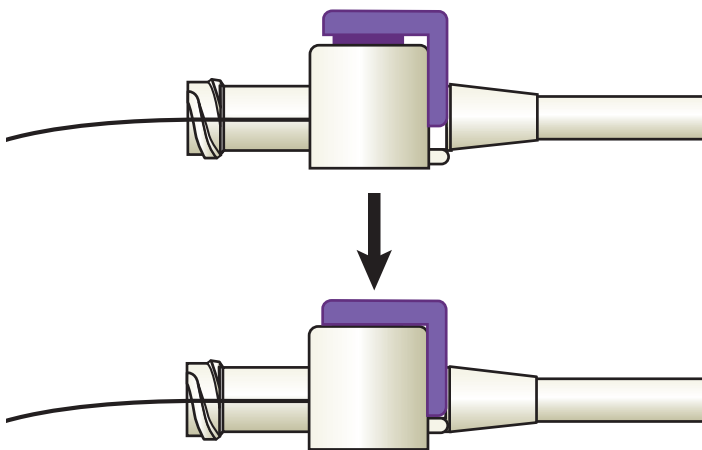

STEP 3: Once placement is confirmed, and the suture locking mechanism has been rotated to the most distal position, press the suture locking mechanism into the hub to secure it. The suture locking mechanism is now locked into position.


UNLOCKING INSTRUCTIONS

OPTION 1:

- Using the Repositioning Tool align the opening of the round section of the Repositioning Tool in line with the handle of the suture locking mechanism.
- Bring the flat back of the Repositioning Tool around the catheter hub.
- Gently squeeze together.
- Remove the Repositioning Tool and rotate the suture locking mechanism to the most proximal position.



OPTION 2: For exchange or removal only, cut the hub off the drainage catheter and sever suture. This will release the suture and the pigtail loop.

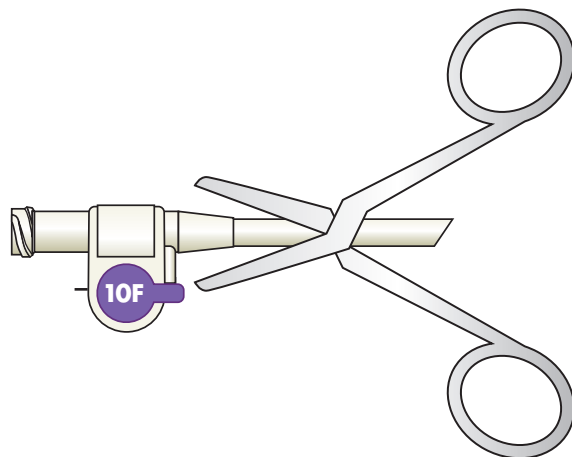

MERITMEDICA®

www.merit.com